SOLARIS

Anodenreiniger Anode Cleaner Nettoie-Anodes

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Instructions for Use Solaris Anode Cleaner

Application: **Solaris Anode Cleaner** is ideally suited for cleaning the anode. The anode should be cleaned with Anode Cleaner after every separation process. The anode is connected to the head of the device with three small plug elements. Pulling the anode downwards lightly will remove it from the socket. Immerse the anode in the Anode Cleaner so that the lattice is completely covered. After the exposure time of approximately 10 minutes, remove the anode and rinse it off thoroughly under running water. Blow the plug and socket elements dry with oil-free compressed air. Repeat procedure if all of soiling has not yet been removed. The anode is plugged back into the device head after cleaning.

NOTE: The Solaris anode cleaner must not be used inside the Solaris unit.

Processing:

Liquid volume:	500 ml
Cleaning time for anode in Anode Cleaner:	approx. 10 min. at room temperature
Bathing motion:	not required

Composition: Solaris Anode Cleaner comprises a solution containing iodine, diluted with water.

Additional information: The Anode Cleaner solution can be reused several times. Renew cleaning solution as soon as cleaning effect is reduced.

Protective measures: Solaris Anode Cleaner may cause impurities on working surfaces and clothing. Comply with normal precautions applying to handling of chemicals. Avoid contact with eyes, skin and clothing. Wear rubber gloves and protective goggles with lateral protection.

Warning: Use anode cleaner outside of device only! Do not allow Anode Cleaner to stand open for longer than 10 minutes. Otherwise, discolorations in the surrounding area may result.

Storage: Do not let container stand open. Close container tightly after use.

Shelf life: See label information.

Disposal: Dispose of product residues in accordance with local regulations.

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DeguDent GmbH