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## Instructions for Use

Solaris Conducting Silver Varnish and Solaris Thinner

**Solaris Conducting Silver Varnish** is required for creating an electrically conducting contact surface on the objects to be electroplated.

Solaris Thinner is used to change the texture of Solaris Conducting Silver Varnish if required. The Solaris Conducting Silver Varnish should only be diluted if strictly necessary, and even then only as much as absolutely required.

**Instructions for processing:** Shake Solaris Conducting Silver Varnish well before use. The integrated mixing ball should be easy to hear while shaking. The surface to be electroplated must be free of dust, clean and completely dry.

Apply **Solaris Conducting Silver Varnish** evenly in a single thin layer all the way to the preparation margin. The silver varnish is connected to the cathode rod and to the electroplating surface by painting the varnish on the cathode (approx. 1 cm all the way around) and a thin strip to the object.

**Warning:** Before electroplating, allow Solaris Conducting Silver Varnish to dry at room temperature for at least 30 minutes (1 hour in the case of non-absorbent surfaces). If this is not observed, the gold bath might be contaminated and the electroplating result compromised.

**Protective measures:** Solaris Conducting Silver Varnish may soil and stain surfaces and clothes.

No special protective measures are required when working with Solaris Conducting Silver Varnish.

**Special risks:** Solaris Conducting Silver Varnish and Solaris Thinner are highly flammable.

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## DeguDent GmbH