Arbeitsanleitung  
Instructions for use  
Mode d'emploi  
Istruzioni d'uso  
Instrucciones de trabajo

**Degussa** 

Multivac® 4

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## A. Equipment and Materials Supplied

- Multivac® 4 complete with  
1 large mixing bowl with cover and stirring mechanism  
1 small mixing bowl with cover and stirring mechanism  
1 suction tube with 2 filter elements  
2 O-rings, small, for the filter elements  
1 bottle of vacuum oil  
1 wrench  
and optionally provided with a  
– wall mounting frame, complete with 1 set of screws, dowel pins, and 1 securing screw,  
**Order Number** (Multivac 4 with wall mounting frame): **2314 0006**  
or with a  
– table stand, complete with 2 securing screws  
**Order Number** (Multivac 4 with table stand) **2314 0005**

In addition, the following can also be supplied:

- Vacuum filling bowl,  
large (Order Number 2350 0009)  
Vacuum filling bowl,  
small (Order Number 2350 0011)  
Mixing bowl,  
extra small (Order Number 2350 0013)  
Mixing bowl,  
extra large (Order Number 2350 0014)

## B. Technical Data

- |                                   |                       |
|-----------------------------------|-----------------------|
| Power voltage:                    | Voltage see typeplate |
| Power frequency:                  | 50 Hz                 |
| Power line fusing:                | 6 A                   |
| Unit fuse in the electric socket: | M 3,15 E              |
| Fuse for electronic circuitry:    | T 0,25 C              |
| Power consumption:                | 310 W                 |
| Vacuum attainable:                | about 10 mbar         |
| Height/width/depth:               | about 610/410/470 mm  |
| Weight without holder:            | about 21 kg           |

A protection device is installed in the Multivac 4 which switches the instrument off in event of overload, and which ensures that the instrument remains switched off until it has cooled down to the permissible operating temperature.

## C. Assembly

1. **Multivac 4 with wall mounting frame**
  1. Hold the wall mounting frame at a suitable location at about shoulder height (round pin to the right), and mark the positions of the four holes to be drilled. Drill these dowel pin holes with a diameter of 12 mm and 45 mm deep, and fasten the mounting frame to the wall.
  2. After the hexagon socket screw has been unscrewed, remove the wooden plate (shipping protection) (2) and keep it.
  3. Suspend the Multivac 4 in the wall mounting frame. In order to secure the Multivac 4 in the proper position, turn the hexagon-head screw (1) supplied with the instrument through the strap at the left under the housing into the base of the housing, and tighten this screw firmly.

For additional assembly instructions see C.II.2.

### II. Multivac 4 with table stand (see picture on title page)

1. After the hexagon socket, screw has been unscrewed, remove the wooden plate (shipping protection) (2) and keep it.
2. Open the black flap (4) and pull the sealing cap on the oil intake fitting upward to remove it.
3. Pour the vacuum oil supplied with the instrument through the nozzle until the oil level has reached the range between the "max." and "min." marks on the indicator (5).
4. Screw the vibrator shaft with the vibrator ball (6) firmly into the threaded connection to the right on the housing.
5. Make the connection to the power line in accordance with VDE regulations (or equivalent local regulations).
6. Plug the suction tube (7) with a filter element into the fitting at the base of the housing (8), and suspend the filter element at the other end of the hose in the clamp on the front panel (9).

The Multivac 4 is now ready for operation.