GB Technical Instructions Instructions for use

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GB

1. multi Safe is a **passive attachment** which counteracts denture dislodgment forces only – all additional forces must be compensated with suitable relief structures and must not allow to displace the denture bridgework. At least two joined abutments with a retentive, cylindrical Interlock and brazing arm at the highest possible level must be used for this purpose. These components must fit together positively and be free of tolerance like a channel-shoulder-pin attachment and stabilize the denture in position in the patient's mouth. Undue denture mobility may cause a fracture of the male part fixed in the mouth.

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2. When fitting a multi Safe, always heed the instructions included in our latest publications.
3. The denture must be checked regularly – at least every six months – including cleaning the mechanism of multi Safe thoroughly and checking that it functions correctly, as well as ensuring that the saddles fit properly (reline if necessary).

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DeguDent GmbH