Rapid Repair

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Directions for Use

Mode d'emploi

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Mode de empleo

D

Rapid Repair

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GB

Rapid Repair

Self-curing repair material for repairing full and partial dentures.

DIRECTIONS FOR USE

Preparing the Denture

- (1) Assemble the broken parts of the denture.
- (2) Cast model.
- (3) Cut back fracture surfaces of the repair to leave a space of 1.5 mm. Bevel the edges to form a shallow V.
- (4) Apply a thin film of 'Vaseline' to the fitting surfaces of the denture taking care to avoid contamination of the area of the repair. Coat model with Isolat and allow to dry for 2 minutes.
- (5) Replace the denture on the model and secure at the periphery with sticky wax.

Mixing

Either of the following techniques may be employed:

- A. Moisten the prepared edges of the broken denture with liquid. Add powder to the moistened area. Repeat the process of adding liquid then powder until the repair is built up in excess of the required thickness.
- B. Mix fluid and powder till obtaining a free-flowing paste. Apply material only in free-flowing state. To avoid run-offs sprinkle with powder. Setting possible in the air or in pressure pot. Polymerisation time 15 min.

Important: Do not use the mix when it fails to a flow freely from the spatula.

Note:

Rapid Repair material should be used at temperatures in the range 21 – 23 °C and it is advisable to condition the mixing vessel to this range, if necessary. As with all acrylic materials the working and setting times increase at lower temperatures and decrease at higher temperatures.

Curing

- (a) In Air
A porosity-free repair is possible by curing undisturbed in air. The setting time at 23 °C is approximately 15 minutes.
- (b) In Pressure Flask
Immerse the repaired denture and model in water at approximately 40 °C. Apply 50 lb/in² (3.5 kg/cm²) pressure and hold for 10 minutes or according to the pressure flask by manufacturer's recommendations.

Finishing

After the material is completely set, remove from the model. Trim and polish in the normal manner.

STORAGE

Store in a cool, dry place (max. 25 °C). Keep containers closed when not in use. Avoid direct sunlight.

SAFETY WARNINGS

Engineering measures

Ensure suitable suction / aeration at the work place and with operational machinery.

Personal protective equipment

Hand protection

Wear protective gloves made of the following materials: butyl rubber.

Eye protection

Close-fitting protective goggles (e.g. closed goggles) recommended

Skin and body protection

Immediately change moistened and saturated work clothes. Use suitable skin protective agents before handling product. Cleanse and apply cream to skin after work. Preventive skin protection recommended.

Do not use in case of hypersensitivity to any of the components of the product.

For dental use only.

The batch number and expiry date are shown on all containers. Please quote both in any communication. Do not use after expiry date.

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