DENSPLY

Special Tray

**Gebrauchsanweisung
Directions for Use
Mode d'emploi
Indicazioni per l'uso
Mode de empleo
Gebruiksaanwijzing
Bruksanvisning
Használati utasítás**

Made in U.K.

50535086 / 5000 / 0503 / KH

DeguDent GmbH
Postfach 1364 · D-63403 Hanau
GERMANY
Telefon +49 61 81/59-50

D

DENSPLY

Special Tray

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DENSPLY

Special Tray

Self-curing acrylic for the preparation of custom-made impression trays. The trays may be used with all types of impression materials.

DIRECTIONS FOR USE

Model Preparation

Prepare a stone or plaster model from a preliminary impression. If a spacer is required use a single sheet of modelling wax adapted over the model and shaped to the required periphery outline. This can be easily removed, without adhering to the tray, prior to finishing.

Proportions

Full tray –

Powder – one level LARGE scoop. approx 20 g

Liquid – up to 12 ml on glass vial

Mixing

Pour the liquid into a convenient polythene or glass vessel and then add the powder. Stir fairly vigorously with a spatula for approximately 45 seconds until a non-sticky dough is formed, which leaves the mixing vessel cleanly.

Adaption

Mould the dough to a convenient form and place in the centre of the template, resting on a polythene sheet. Place another polythene sheet on the top of the dough and roll or apply pressure to the dough so that it completely fills the template. Remove the top polythene sheet and trim the excess material from the periphery of the template. Carefully remove the flattened material from the template and locate on to the prepared model. With light finger pressure contour it to shape. Start from the palatal region and work towards the periphery. Avoid the application of excess pressure at this stage.

Note:

Should the material rupture during shaping over high spots, such as a tooth, immediately apply a small piece of the excess material and recontour before setting occurs. Trim the excess from around the periphery of the model with a sharp knife – do not remove from model but recontour with light finger pressure and smooth out any surface wrinkles.

To form a handle, shape excess material into an oblong strip. Apply a little liquid to the contact area, attaching the handle to the tray, hold for a few seconds and then mould to shape.

Finishing

Remove the tray from the model when it has cooled following the curing exotherm and is sufficiently rigid not to distort. When finally set (in approximately ten minutes at 23 °C) remove the wax spacer and trim around the tray periphery with a bur or stone to remove excess material and provide a smooth surface.

STORAGE

Store in a cool, dry place (max. 25 °C). Keep containers closed when not in use. Avoid direct sunlight.

SAFETY WARNINGS

Engineering measures

Ensure suitable suction / aeration at the work place and with operational machinery.

Personal protective equipment

Hand protection

Wear protective gloves made of the following materials: butyl rubber.

Eye protection

Close-fitting protective goggles (e.g. closed goggles) recommended

Skin and body protection

Immediately change moistened and saturated work clothes. Use suitable skin protective agents before handling product. Cleanse and apply cream to skin after work. Preventive skin protection recommended.

Do not use in case of hypersensitivity to any of the components of the product.

For dental use only.

The batch number and expiry date are shown on all containers. Please quote both in any communication. Do not use after expiry date.

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