Stellon QC-20

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Directions for Use

Mode d'emploi

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Mode de empleo

Made in U.K.

DIN EN ISO 1567: 2000
Typ I, Gruppe 1

DeguDent GmbH

Postfach 1364 · D-63403 Hanau
GERMANY
Telefon +49 61 81 / 59-10

50535088 / 5000 / 0503 / KH

D

Stellon QC-20

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GB

Stellon QC-20

Fast curing acrylic for aesthetic, full and partial dentures. Stellon QC-20 is according to DIN EN ISO 1567: 2000, Type I, Class 1.

DIRECTIONS FOR USE

Waxing and investing

Follow normal procedure for gypsum models.

Wax removal

Immerse the flask in boiling water for five minutes. Remove and open the flask, lift out wax and flush thoroughly with boiling water to which detergent solution has been added, followed by clean boiling water. Allow to cool.

Apply Isolant

Use a clean brush to apply Isolant to all plaster surfaces. Absorb excess from around teeth while it remains fluid, and make quite sure there is non on the exposed surface of the teeth. Allow to dry before packing.

Mix

Recommended ambient temperature for mixing and packing is 23 °C. Pour liquid (Universal Denture Liquid) into a mixing vessel and add appropriate powder in a slow steady stream, until an excess appears on the surface. Mix Powder/Liquid Ratio 23 g : 10 ml. Hold the vessel in the hand, tap 3 or 4 times to bring any excess liquid to the surface and add sufficient powder to absorb this liquid. Invert the vessel and discard any surplus powder. Thoroughly spatulate the mix for 1 minute. Cover the vessel and wait for the mix to reach the correct stage for packing and pressing (dough time) 12–16 min. This correct stage is reached when the mix can be separated clearly from the walls of the mixing vessel with a spatula.

Pressing

Pack the material into the mould, ensuring that the temperature of the flask is approximately 23 °C, and cover with a thin polythene sheet. Assemble the flask and close slowly under a press. Open the flask, paint the model half of the flask with Isolant, remove separating sheet and cut away the flash.

Reassemble the flask, close under press and transfer to a 'Stellon' Security flask clamp.

CAUTION

Universal Denture Liquid must be used in conjunction with Stellon QC-20.

RECOMMENDED CURES

CURE A: Immerse the clamped flask in boiling water, continue heating to regain boiling temperature and boil for 20 minutes.

CURE B: Immerse the clamped flask in boiling water, turn off the heat for 20 minutes, then continue boiling for another 20 minutes.

Alternative cures

Other longer cure techniques can, if desired, be employed with all DENTSPLY Denture Base Materials.

Note

After all curing cycles, bench cool the flask for 10 minutes prior to immersing in cold water before deflasking. Finish and polish in the usual manner.

REPAIRS

To obtain maximum repair strength, it is recommended that repairs or additions be made using the denture base that was employed initially and using one of the recommended curing cycles. An autopolymerising resin such as DENTSPLY Rapid Repair may also be used for this purpose.

STORAGE

Store in a cool, dry place (max. 25 °C). Keep containers closed when not in use. Avoid direct sunlight.

SAFETY WARNINGS

Engineering measures

Ensure suitable suction / aeration at the work place and with operational machinery.

Personal protective equipment

Hand protection

Wear protective gloves made of the following materials: butyl rubber.

Eye protection

Close-fitting protective goggles (e.g. closed goggles) recommended

Skin and body protection

Immediately change moistened and saturated work clothes. Use suitable skin protective agents before handling product. Cleanse and apply cream to skin after work. Preventive skin protection recommended.

Do not use in case of hypersensitivity to any of the components of the product.

For dental use only.

The batch number and expiry date are shown on all containers. Please quote both in any communication. Do not use after expiry date.

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